**ENGLISH**
**GENERAL INFORMATION**

**GENERAL INFORMATION**
3M™ ESPE™ Clinpro™ White Varnish is a fluoride-containing varnish for application to enamel and dentin for the treatment of hypersensitive teeth. Clinpro White Varnish is moisture and saliva-activated. It will spread on and adhere to moist teeth. Clinpro White Varnish is an alcohol-based solution of modified resin. The product is sweetened with xylitol. Clinpro White Varnish contains an innovative tri-calcium phosphate (TCP) ingredient exclusively from 3M ESPE. Calcium and phosphate ions are naturally occurring components of saliva long associated with maintaining healthy teeth.

Mint flavor is supplied in unit-dose packages containing 0.50 ml of Clinpro White Varnish. Each 0.50 ml of Clinpro White Varnish contains 25 mg sodium fluoride equivalent to 11.3 mg fluoride ion.

**INDICATIONS**

- Use as a fluoride-containing coating that:
- Treats hypersensitive teeth
- Treats exposed dentin and root surface sensitivity

**CONTRAINDICATIONS**

Ulcerative gingivitis and stomatitis

**PRECAUTIONARY INFORMATION for patients**

To limit ingestion, determine the proper application amount for the patient. Discontinue use of other prescriptive fluoride preparations (i.e. gels, trays) for 24 hours following Clinpro White Varnish application. Children taking fluoride supplements should discontinue use of supplements for 2-3 days following treatment.

**PRECAUTIONARY INFORMATION for dental personnel**
3M ESPE MSDS information can be obtained from www.3MESPE.com or contact your local subsidiary.

**HAZARD INFORMATION**

This product as sold is a flammable material. Once the material has adhered to the teeth, this product is no longer flammable.

**INSTRUCTIONS FOR USE**

**General application instructions:**

- The tooth surface(s) to be treated should minimally be “toothbrush” clean. A prophylaxis is not required.
- Open the unit-dose package of Clinpro White Varnish and dispense the entire content onto the round application guide on the back of the pouch, or onto one of the application guide stickers provided in the box.
- IMPORTANT: Use the applicator brush to thoroughly mix Clinpro White Varnish since components of all sodium fluoride varnishes can separate during storage. While mixing, keep the material evenly distributed inside the inner circle of the application guide.

- One or two brush loads of Clinpro White Varnish may be sufficient for an entire quadrant when used for routine application in recall visits. For more extensive treatments, refer to the maximum amounts suggested in the application recommendations below.

- Use the application guide by applying the material within the upper half of the circle down to the line that indicates the maximum amount.
- Maximum Application Guidelines:**

**0.25 ml dosage** (apply material from the top of the inner circle to the line that indicates 0.25 ml)

- To be used to treat patients with primary dentition.

- To be used to treat any patient requiring limited tooth surface coverage.

**0.40 ml dosage** (apply material from the top of the inner circle to the line that indicates 0.40 ml)

- To be used to treat patients with mixed dentition who require extensive tooth surface coverage.

DO NOT use this amount to treat patients with primary dentition only. Follow guidelines above.

**0.50 ml dosage** (apply material from top of the inner circle to the line that indicates 0.50 ml)

- To be used to treat patients with permanent dentition who require extensive tooth surface coverage.
- DO NOT use this amount to treat patients with primary dentition only. Follow guidelines above.

**NOTE:** If the full amount (0.50 ml) is being used, the Clinpro White Varnish does not need to be dispensed on the application guide. A mix pad or other surface may be used.

- Clinpro White Varnish will set to dentition in the presence of saliva. For best results, excessive saliva should be removed prior to application by blotting the tooth surface(s). To prevent premature setting of varnish on bristles, avoid rinsing or brushing with toothpooled saliva. Apply Clinpro White Varnish evenly in a **thin** layer over treatment areas (FA). For larger surface areas, apply Clinpro White Varnish in sweeping horizontal brush strokes across multiple teeth.

- After application, instruct the patient to close his or her mouth to set the Clinpro White Varnish. Rinsing or suctioning immediately after application is not recommended. You may see a thin coating on the teeth. The patient may feel the thin coating when rubbing the treated area with his or her tongue.

- The treatment period for Clinpro White Varnish is minimally 4 hours, preferably overnight.

- During the treatment period, instruct the patient to avoid hard and sticky foods, including alcohol (beverages, oral rinses, etc.), hot beverages, toothbrushing, and flossing.

- After the treatment period, the patient can remove Clinpro White Varnish by brushing and flossing, or wait until the coating naturally wears away (within 24 hours).

**STORAGE AND USE**

This product is designed to be stored and used at room temperature. Shelf life at room temperature is 24 months. Ambient temperatures routinely higher than 27°C/80°F, or lower than 10°C/50°F, may reduce shelf life. See outer package for expiration date.

**DISPOSAL**

See the Material Safety Data Sheet (available at www.3MESPE.com or through your local subsidiary) for disposal information.

**CUSTOMER INFORMATION**

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

**WARRANTY**
3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

**LIMITATION OF LIABILITY**

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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