# FD 300 top wipes

DE	Flächendesinfektion	CS	Dezinfekce ploch
EN	Surface disinfection	SK	Dezinfekcia plôch
FR	Désinfection des surfaces	SL	Površinsko razkuževanje
IT	Disinfezione di superfici	HR	Dezinfekcija površina
ES	Desinfección de superficies	HU	Felületi fertőtlenítés
PT	Desinfecção de superfícies	EE	Pindade desinfitseerimiseks
NL	Oppervlakdesinfectie	LV	Virsmu dezinfekcija
SV	Ytdesinfektion	LT	Paviršių dezinfekcijos priemonė
FI	Pintojen desinfiointiaine	RU	Дезинфекция поверхностей
NO	Overflatedesinfeksjon	BG	За дезинфекция на повърхности
DA	Overfladedesinfektion	RO	Dezinfecția suprafețelor
EL	Απολύμανση επιφανειών	TR	Yüzey Dezenfeksiyonu
PL	Dezynfekcja powierzchni		

DÜRR DENTAL AG  
Höpfigheimer Straße 17  
74321 Bietigheim-Bissingen, Germany  
Tel.: +49 7142 705-0  
Fax: +49 7142 705-500  
info@duerr.de · www.duerr.de

CE 0297

 DÜRR  
DENTAL

Z.300-2015-10

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Feldstraße 7a, 6020 Innsbruck, Austria ☎ + 43 512572834

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Seestraße 108, 9326 Horn, Switzerland ☎ + 41 718441030

FD 300 top wipes are ready-to-use alcohol-free wipes which simultaneously clean and disinfect the surfaces of non-invasive medical devices such as treatment units, medical devices, etc. FD 300 top wipes have an intensive cleaning action and a pleasant fragrance. They are highly compatible with a variety of surfaces. Extra large wipes measuring 175 x 250 mm.

**Composition:** FD 300 top wipes contain a ready-to-use solution of FD 300. 100 g solution of active substance contain 0,36 g of 3-aminopropyl-dodecyl-1.3-propane diamine, 0,21 g of didecyl-methyl-poly(oxyethyl)-ammonium propionate, non-ionic tensides, alkaline cleansing compounds, complexing agents, auxiliary substances and water.

**Spectrum of activity:** FD 300 – bactericidal, tuberculocidal, fungicidal, virucidal (enveloped viruses, incl. HBV, HCV and HIV<sup>1)</sup>, as well as naked viruses such as adenoviruses, polyoma SV-40 viruses, polio viruses, noro viruses).

**VAH/DGHM list.** Tested in accordance with EN 13727, EN 13624, EN 14348, EN 14476, EN 14563. EN 13697.

<sup>1)</sup> In accordance with the recommendations of the RKI (Robert Koch Institute) (Bundesgesundheitsbl. 47, 62-66, 2004).

**Directions for use:** Wet surfaces. Exposure time is 2 min. in accordance with VAH/DGHM. For the inactivation of viruses we recommend 5 min. For further information please also see product information on the Internet.

**Caution:** FD 300 top wipes are not suitable for aluminium surfaces.

see imprint.

---

Wear protective gloves. If skin irritation or rash occurs: Get medical advice/attention. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

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