Lunos® Prophyxapulver Gentle Clean

Lunos® Prophy powder Gentle Clean

urochemie GmbH + Co. KG
Max-Planck-Str. 27, 72086 Kornwestheim
Vertrieb / Distribution: DÜRRENTAL AG
Höfigheimer Straße 17
74321 Bietigheim-Bissingen, Germany
Tel: +49 7142 705-0
Fax: +49 7142 705-50
info@duer.de www.duerdental.com
Made in Germany
ZIFRO04.020.16.12

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### Lunos® Prophyxapulver Gentle Clean

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## EN Operating Instructions

**Lunos® Prophy Powder Gentle Clean**
The following versions of Prophyx Powder are available:

**Spearmint (REF: CP2630A2250)**, contains limonene
**Orange (REF: CP2630A2250)**, contains limonene
**Neutral (REF: CP2610A2250)**, contains no flavouring agents

**Notice:** Prophyx Powder Gentle Clean is a medical device. This product is only intended for professional dental applications and must only be used by trained personnel.
Please refer to the instructions for use of the supragingival applicator to powder for the proffissi Gentle Clean Neutral.

**Warning:** Prophyx Powder Gentle Clean is designed for supragingival application on the surface of the tooth using commercial powder jet devices. Please note the operating instructions of the powder jet device before use of the prophyx powder.

### Application

This product is recommended for:

- Supragingival cleaning of the tooth surface
  - Removal of discoloration and plaque
- Preparation and cleaning of the tooth surface before:
  - Adhesive attachment of orthodontic brackets
  - Application of fluoride and bleaching preparations
  - Insertion of composite restorations, fixed dentures, e.g. inlays, onlays, crowns, veneers etc.
  - Application of sealing materials

**Contraindications**
Patients suffering from chronic airway diseases (e.g. bronchitis or asthma) must not be treated with a powder-water jet device since the powder present in the aerosol can lead to difficult breathing. Likewise, the product should not be used on non-sensitized enamel and hypominalized tooth enamel and „white spots“.

In certain cases, flavouring agents present in the powder may cause skin irritation or allergic reaction. For this reason, patients with known allergies to flavouring agents should not be treated with „Prophyx Powder Gentle Clean Neutral“. Please make sure not to direct the powder jet device directly at fixed dental restorations and fillings as this may damage the restorations.

**Warning:** Prophyx Powder Gentle Clean is designed for supragingival use exclusively. Please make sure to use „Prophyx Powder Gentle Clean“ in powder jet devices for supragingival application exclusively.

**Preparation:**
Please remove all residual powder from the powder chamber of the powder jet device before refilling it with Prophyx Powder Gentle Clean

• Shake the dosette bottle before removing any Prophyx Powder Gentle Clean. Please remove the seal from the bottle for this purpose. To do this, pull the cap off the bottle, pull off the seal and then screw the lid back on the bottle.

• Fill the powder prophyx powder chamber, make sure that the chamber is completely dry. There must be no residual moisture in the chamber, e.g. from reprocessing measures.

• After complete of powder chamber, you need to make sure not to exceed the maximum filling level of the powder chamber.

• Fold the „rocker“ all the way up or unscrew the lid from the bottle. To minimise the risk of embolism from air being introduced into the blood circulation.

• Use the „rocker“ for this purpose and to slightly compress the bottle.

### Recommendations for treatment

Patient and operator need to wear suitable eye protection, e.g. protective goggles.

The operator needs to wear suitable protective equipment in the form of mouth and nose protection and suitable protective gloves.

Use a suitable spray mist aspiration canula, e.g. the Dürr Dental Prophyx Can-Arma® (REF: CP2610A2250), sans aromastoffe

Do not direct the spray nozzle at the gingiva during the treatment. Direct exposure of the soft tissue may damage the gingiva. The powder jet nozzle must not be aimed directly into the gingival pocket as there is a risk of erythema arising in the tissue and/or of embolism from air being introduced into the blood circulation.

To protect sensitive skin, please apply Vaseline to the patient's lips before commencing the treatment.

After completion of treatment, clean the patient's face with a moist disposable cloth. This prophyx powder cannot be used before treatment.

**General Information**
This prophyx powder does not have a cariogenic effect.
Please make sure to always remove any residual powder from working surfaces, front surfaces of cabinets or treatment units using a moist cloth.

Notes for the patient
Please advise your patient that the ingestion of food or beverages (tea, coffee or other food items) within the first two to three hours after completion of the treatment can cause renewed discoloration of the teeth.

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