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
Lunos® Prophylaxepulver Perio Combi
Lunos® Prophylaxis powder Perio Combi

Rochechemie GmbH • Co. KG
Max-Planck-Str. 27, 70806 Kornwestheim
Vertrieb/Distribution: DÜRR DENTAL AG
Höfheimerer Straße 17
74321 Biegelheim-Bissingen, Germany
Tel: +49 7142 705-0
Fax: +49 7142 705-500
info@duer.de • www.duer.de
Made in Germany

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### Lunos® Prophylaxepulver Perio Combi

**Prophylaxepulver Perio Combi** (REF: CPZ640A1950)

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### Lunos® Prophylaxis Powder Perio Combi

**Prophylaxis Powder Perio Combi** (REF: CPZ640A1950)

**Notice:** Prophylaxis Powder Perio Combi is a medical device. This product is only intended for professional dental applications and must only be used by trained specialists!

\*Prophylaxis Powder Perio Combi\* is designed mainly for sub- and supra-gingival application in commercial powder jet devices. Please note the operating instructions of the powder jet device before use of the prophylaxis powder.

**Application**
This product is recommended for:

- Removal of periodontal biofilm in the scope of periodontal maintenance therapy (recall therapy) after completed initial therapy for a pocket depth of up to 10 mm.
- Implant cleaning and treatment of peri-implantitis after completed initial therapy
- Gentle removal of plaque at the sulcus and particularly sensitive dental services.
- Removal of slight discolorations in supragingival regions
- Removal of plaque on orthodontic appliances (brackets)

#### Contraindication

Patients suffering from chronic airway diseases (e.g. bronchitis or asthma) must not be treated with a powder-water jet device since the powder present in the aerosol can lead to difficult breathing. Likewise, the product should not be used on non-intact enamel and hyperminalised tooth enamel and "white spots". Please take into consideration that the treatment of deep periodontal pockets can lead to bacteremia. For this reason, suitable measures must be undertaken before treating a risk patient (endocarditis patient, pregnant and lactating women, infectious diseases, immunodeficiency, neutropenia, agranulocytosis, diabetes, haemophilia, patients on treatment; Radiation therapy patients, chemotherapy patients and patients during antibiotic therapy). Please make sure not to direct the jet of the powder jet device directly at fixed dental restorations and fillings as this may damage the restorations.

**Preparation**
Please remove all residual powder from the powder chamber of the powder jet device before refilling with powder. Please use the powder jet device directly at fixed dental restorations and fillings as this may damage the restorations.

- Shake the closed bottle before removing any Prophylaxis Powder Perio Combi.
- Please remove the seal on the bottle before removing the powder. For this purpose, take the lid off the bottle, pull off the seal and then screw the lid back on the bottle.
- Before filling the powder chamber, make sure that the chamber is completely dry. There must be no residual moisture in the chamber, e.g. from repressing fillings.

Fill the powder into the powder chamber, you may take care not to exceed the maximum filling level of the powder chamber.

Fold the "rock" all the way up or unscrew the lid from the bottle. To minimise the amount of dust generated when you fill the chamber, use the "rock" to use the "rock" for this purpose and to compress the bottle only slightly.

#### Recommendations for treatment

Patient and operator need to wear suitable eye protection, e.g. protective goggles. The operator needs to wear suitable protective equipment in the form of mouth and nose protection, apron, gloves and suitable protective glasses. Use a suitable spray mist aspiration cannula, e.g. the Dürr Dental Prophylaxis Canula.

Do not direct the spray nozzle directly at the gingiva during the treatment. Direct exposure of the soft tissue may damage the gingiva. Also there's a risk of embolism arising in the tissue or of embolism from air being introduced into blood circulation.

To protect sensitive skin, please apply Vaseline to the patient's lips before commencing the treatment.

After completion of treatment, clean the patient's face with a moist disposable cloth. This prophylaxis powder cannot be used remove tartar.

This prophylaxis powder does not have a cariogenic effect. Please make sure to always remove any residual prophylaxis powder from working surfaces, front surfaces of cabinets of treatment units using a moist cloth.

#### Notes for the patient

Please inform the patient that the ingestion of food or beverages (tea, coffee or other food items) within the first two to three hours after completion of the treatment can cause discoloration of the teeth.

Use the product in a well-ventilated place at a temperature not above 25°C. The shelf life of the product is 30 months.

DÜRR DENTAL (Products) UK Ltd.
41 Linwell Way, Feltham, Middlesex, UK
Tel: +44 (0)181 6082660
Kering, Norfolk, Norfolk NG16, United Kingdom ☎ +44 1536526740

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