# NUPRO<sup>®</sup> SENSODYNE<sup>®</sup>

## PROPHYLAXIS PASTE

With NovaMin<sup>®</sup>

These instructions cover all NUPRO Sensodyne Prophylaxis Paste Flavors/Grits.

NUPRO Sensodyne is a premium Prophylaxis Paste containing NovaMin<sup>®</sup>, a calcium sodium phosphosilicate. The dye-free formulations are available in the following two grits to gently remove dental plaque and stains. It is available in fluoride and non-fluoride formulations.

Polish Grit

Ideal for younger patients and proph procedures requiring minimal tooth abrasion.

Stain Removal Grit

Suitable for most cleaning procedures where a high level of polish is desired for stain removal.

## INDICATIONS FOR USE

NUPRO<sup>®</sup> Sensodyne<sup>®</sup> Prophylaxis Paste with NovaMin<sup>®</sup> is intended for cleaning and polishing procedures, prior to and after scaling and root planing as part of a professionally administered dental prophylaxis treatment. Secondly, NUPRO<sup>®</sup> Sensodyne<sup>®</sup> Prophylaxis Paste with NovaMin<sup>®</sup> can be used for the immediate relief of tooth sensitivity and for lasting sensitivity relief for up to 4 weeks (28 days) after just one application.

## CONTRAINDICATIONS

- Not for use on patients with a known allergy to any of the components.
- Not for use on restorations. Use NUPRO<sup>®</sup> Shimmer<sup>®</sup> Polishing Paste for Restorations.

## INGREDIENTS

Medicinal Ingredients: 2.72% Sodium Fluoride (1.23% fluoride ion) (for fluoride containing pastes only)

Non-Medical Ingredients (for all pastes): Glycerin, Pumice, 15% Calcium Sodium Phosphosilicate (NovaMin<sup>®</sup>), Sodium Silicate, Titanium Dioxide, Methyl Salicylate, Purified Water, Flavor, Sodium Carboxymethylcellulose, Sodium Saccharin.

## WARNINGS

- Avoid prolonged contact with inflamed tissue.
- Avoid eye contact. If contact with eyes occur, immediately flush eyes with copious quantities of water for 15 minutes. Seek medical attention if irritation or discomfort persists.
- Avoid using fluoride-containing pastes in individuals with hypersensitivities to fluoride as adverse reactions can occur. In the case of adverse reactions, seek medical attention immediately.
- Unit dose cups contain enough proph paste for a single procedure. Each cup is intended to be used once for a single patient. Reuse of an unfinished cup for another patient could result in cross-contamination.
- Reuse of contaminated instruments to remove paste from multi-use jars can result in cross-contamination.

## PRECAUTIONS

- Prophylaxis pastes containing fluoride are not considered a replacement for other forms of fluoride therapy.
- Prophylaxis Pastes are to be used only by individuals professionally trained to perform a dental prophylaxis.
- During use, wear protective glasses and gloves.

## ADVERSE REACTIONS

For fluoride containing pastes, the following adverse reactions are possible in individuals hypersensitive to fluoride: salivation, gastric distress, headache, weakness, tremors, spasms, convulsions, eczema, atopic dermatitis or urticaria.

## RESIDUAL RISKS

- Using the paste on a patient with inflamed tissue could cause further irritation.
- Failing to wear protective glasses can result in eye irritation on contact with the paste.

## ENGLISH

- Applying fluoride-containing pastes to individuals with fluoride hypersensitivities can result in serious illness.
- Reactions may occur in hypersensitive individuals or in individuals with allergies to any of the raw materials of the paste.
- Use on restorations may affect the integrity of the restorative material and could lead to caries and/or periodontal disease.
- Cross-contamination can lead to patient infection or illness.

## STEP-BY-STEP INSTRUCTIONS

When using bulk jars, sufficient paste for one procedure should be removed and placed in a clean dappen dish or other suitable container. Close lid immediately after removing paste. When using single-use cups, insert the cup into a Nupro Prophy Paste Holder for easier handling.

Using a disposable prophy angle or a conventional metal prophy angle, fill the cup by pressing the face of the cup into the paste while running the angle at low speed.

Apply paste and leave in contact for 60 seconds. Throughout the procedure, sufficient paste should be maintained on the cup to avoid overheating of the tooth surface. Saliva evacuation is recommended to avoid excessive dilution and swallowing of the paste.

Rinse full mouth with water spray. For any white residue remaining following normal rinsing, have the patient swish water or mouth rinse around the gingiva. Polishing prior to scaling is another option to reduce the white residue on the gingiva.

## STORAGE

When using the bulk jars, close lid immediately after use to avoid moisture loss. Do not store above 25 C (77 F). Keep out of direct sunlight.

## EXPIRATION DATE AND LOT NUMBER

The ReOrder Number, Lot Number, and Expiration Date should be referenced in all correspondence requiring identification of the product. Do not use after expiration date.

## SAFETY DATA SHEET

is available on our website, www.dentsplysirona.com, or by contacting Customer Service at 1-800-999-8826.

## SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

All SSCP documentation can be found on the Eudamed database.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the Competent Authority of the Member State.

Manufactured by: Dentsply Professional 1301 Smile Way, York, PA 17404 USA www.dentsplysirona.com

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